



**MicroLite™**  
Portable  
Massage Chair

Congratulations on the purchase of your new MicroLite™ portable massage chair by Stronglite. With proper care, your chair should provide many years of healing.


### Set up and Adjustment

- Remove chair from box and/or carry case.
- Unfold front legs to open chair.
- Lift seat and place into one of three seat adjustment positions, as appropriate.
- Release headrest locking arm and unfold headrest. Adjust the headrest height and angle for maximum comfort. Lock by pushing down on the locking arm.
- Lift armrest and firmly place armrest locking arm into one of three armrest adjustments positions.
- Loosen chest support knob to adjust height of chest pad, as needed. Re-tighten knob.
- Adjust arm, seat and headrest positions, as necessary.

### Breakdown and Storage

- Lift armrest and lower into down position.
- Release headrest, lower, and re-lock.
- Release seat from adjustment bracket and fold seat down against knee pad.
- Slide front legs to closed position.
- Store in carry case.

### PRODUCT OVERVIEW

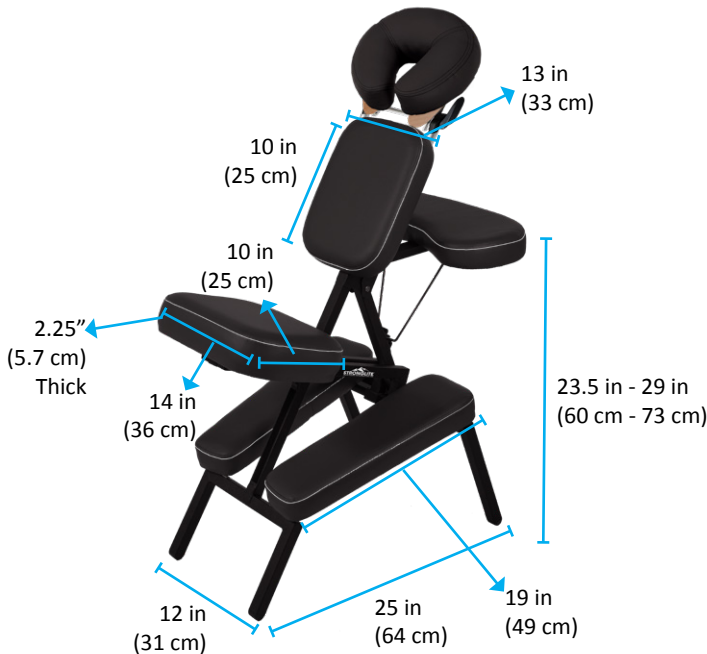

- Product overall dimensions: 41.5" x 19.5" x 9"
- Product weight: 16.5 lbs

### IMPORTANT

- Beware of pinch points when setting up and breaking down the chair.
- Make sure that the seat, armrest, and headrest are securely fastened before seating your client.
- The MicroLite chair is exceptionally strong and stable. However, it is not designed for use with clients who weigh over 300 lbs.
- Use only mild soap and warm water in the cleaning of your upholstery, using a non-abrasive sponge or rag. For disinfecting, or tougher stain, we recommend a 1:10 bleach/water-based solution or one of the cleaning and disinfecting products listed on our website. Do not use Armor All™, acetones, or citrus based solvents as they may damage your vinyl.
- Avoid storing your chair in extreme temperature and humid environments, especially car trunks. We recommend the use of a carry case for storage.
- Check screws and bolts periodically, tightening as necessary. A drop or two of a silicone based lubricant should alleviate any squeaking that may occur during the long lifetime of the product.

### LIFETIME LIMITED WARRANTY

Earthlite guarantees our Stronglite portable massage chairs, excluding the foam and vinyl, against defects in material and workmanship for as long as you, the original purchaser, own the chair, or for five years after the model is discontinued. The foam and vinyl are warranted for three years from the original date of purchase. This warranty does not cover damage caused by misuse, accidents, or neglect, which includes tears in the upholstery and damage caused by harsh chemical-based cleaners. This warranty is limited to product repair or replacement, at Stronglite's sole discretion. You are responsible for shipping the product back to the factory for any warranty claim. Earthlite will return the warranted product, shipping prepaid, to customers in the continental USA. International customers are responsible for all shipping charges (inbound and outbound), as well as any applicable duties, and taxes.

### CONTACT US

Stronglite® by Earthlite  
990 Joshua Way  
Vista, CA. 92081

800-872-0560  
760-599-1112

info@earthlite.com

Visit us online at  
[www.earthlite.com](http://www.earthlite.com)